



TECHNICAL INFORMATION

C2U-TAURUS NERO / MAC2

MAC
MY ACTIVE CARE

Therapeutic shoe with rigid midsole that avoids forefoot hyper-loading. Not equipped with insole as it is predisposed exclusively to accommodate custom insoles. The **MAC2** lines, if used for containing of customized plantar insoles released under medical indication, made and controlled applying standard tests by a licensed healthcare professionals, are medical devices for the treatment of the foot morphology alterations

MAC footwear is in compliance with the International Guidelines of the IWGDF for the management of the diabetic foot. <https://iwgdfguidelines.org/guidelines/guidelines>.

THERAPEUTIC INDICATIONS

Diabetic foot a risk 3 (with loss of protective sensation and/or peripheral arterial disease and/or foot deformity and one or more of the following: history of a foot ulcer, a lower-extremity amputation - minor or major, end-stage renal disease).

Arthrodesis of the toes.

Arthrodesis of the tarsus.

Charcot foot, or biomechanical problems of the movement (high footwear MAC 2).

TECHNICAL CHARACTERISTICS

OUTSOLE: EVA ULTRALIGHT WITH RUBBER INSERTS

MIDSOLE: RIGID TYPE

INSOLE: REMOVABLE PREFORMED INSOLE

HIGH HEEL LAST MEASURE: 10 MM

OUTSOLE DROP MEASURE: 5 MM

UPPER: LEATHER

VAMP & LATERAL SIDE UPPER: ELASTICIZED LEATHER

LINING: ASSEMBLED LYCRA TEXTILE

COLLAR FOAM: FOAM RUBBER

BACK COUNTER: MOLDED NYLON

BACK COUNTER FOAM: FOAM RUBBER

REINFORCEMENTS: GLASS WOOL

HARD-TOE END: REBOUND EFFECT THERMO NYLON

LACING SYSTEM: DOUBLE ADJUSTABLE VELCRO

AVERAGE SHOE WEIGHT SIZE 41: Gr.510

SIZE - FITTING: FIT 13

PRODUCT CLEANING

The cleaning of the product must be carried out using:


- for the upper in fabric, use a cloth slightly dampened with water and neutral soap, rub gently and dry;
- for leathers: special creams (moisturizing-polishing) specific for the type of leather to be treated and compliant and approved by the EEC, coated and subsequently polished with a clean, soft and non-abrasive damp cloth;
- for suede or similar: use only brushes or soft cloths and only with dry skin to remove any stains (mud or similar);
- make sure to perform a regular hydration of the leather by applying special creams, especially in the case of particularly soft leathers such as nappa, deer etc.
- do not use alcoholic and corrosive liquids, detergents, alcohol, laundry detergents, bleach, ammonia and solvents;
- do not use abrasive materials and/or products brushes, steel wool, glass paper, blades or similar, to remove stains or residues;
- do not machine wash;
- do not soak the product.

The footwear **MUST NOT** be machine washed or washed by being immersed in or put under water. To clean the lower lateral surface of the sole use soft sponges dampened in warm water not exceeding 30° C, and neutral soap. Do not dry the wet or soaked product in front of heat sources. If the product gets accidentally wet, let it dry away from heat sources and in a dry environment. Optima Molliter Srl shall **NOT** be held liable for damages caused by abrasive products, by products which attack the materials, by direct exposure to heat sources and by improper use/maintenance.

Class I medical device according to Regulation (EU) 2017/745

SIZES

	40	41	42	43	44	45	46	47
A	27,1	27,8	28,5	29,2	29,9	30,6	31,3	32
B	9,3	9,5	9,7	9,9	10,1	10,3	10,5	10,7
C	27	27,5	28	28,5	29	29,5	30	30,5

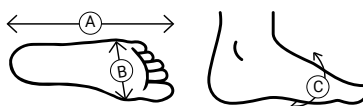

Optima Molliter Srl

Company Certified under: ISO 9001:2015 IT319043 - ISO 13485:2016 IT319042
Via Breda 19/21 62012 Civitanova Marche (MC) Italy T +39 0733 801060
info@optimamolliter.com / www.optimamolliter.com

