



TECHNICAL INFORMATION

C3U-BRIAN NERO / MAC3

MAC
MY ACTIVE CARE

Protective footwear with propulsive semi-rigid midsole that redistributes plantar pressure loads. The **MAC3** lines, if used for containing of customized plantar insoles released under medical indication, made and controlled applying standard tests by a licensed healthcare professionals, are medical devices for the treatment of the foot morphology alterations

MAC footwear is in compliance with the International Guidelines of the IWGDF for the management of the diabetic foot. <https://iwgdfguidelines.org/guidelines/guidelines>.

THERAPEUTIC INDICATIONS

Diabetic foot at risk 1 (foot with loss of protective sensitivity or peripheral arterial disease).

Biomechanical problems of the movement.

Metatarsal pain, with a customized plantar insole made by a licensed healthcare professional; If associated with a customized plantar insole MAC3 is indicated for the management of diabetic foot:

- diabetic foot at risk 2 (foot with loss of protective sensitivity associated with deformity or foot with peripheral arterial disease associated with deformity);
- diabetic foot at risk 3 (foot with previous ulcerations) at risk of remission.

TECHNICAL CHARACTERISTICS

OUTSOLE: EVA WITH RUBBER INSERTS AND INSIDE NYLON INSERT

MIDSOLE: SHOCK ABSORBING FOAM INSOLE

INSOLE: MEMORY FOAM

HIGH HEEL LAST MEASURE: 15 MM

OUTSOLE DROP MEASURE: 8 MM

UPPER: LEATHER

LINING: SUEDE TRANSPIRANT SOFT TEXTILE

COLLAR FOAM: FOAM RUBBER

BACK COUNTER: MOLDED NYLON

BACK COUNTER FOAM: FOAM RUBBER

REINFORCEMENTS: GLASS WOOL

HARD-TOE END: REBOUND EFFECT NYLON

LACING SYSTEM: DOUBLE ADJUSTABLE VELCRO

AVERAGE SHOE WEIGHT SIZE 41 Gr. 350

SIZE - FITTING: FIT 11

PRODUCT CLEANING

The cleaning of the product must be carried out using:


- for the upper in fabric, use a cloth slightly dampened with water and neutral soap, rub gently and dry;
- for leathers: special creams (moisturizing-polishing) specific for the type of leather to be treated and compliant and approved by the EEC, coated and subsequently polished with a clean, soft and non-abrasive damp cloth;
- for suede or similar: use only brushes or soft cloths and only with dry skin to remove any stains (mud or similar);
- make sure to perform a regular hydration of the leather by applying special creams, especially in the case of particularly soft leathers such as nappa, deer etc.
- do not use alcoholic and corrosive liquids, detergents, alcohol, laundry detergents, bleach, ammonia and solvents;
- do not use abrasive materials and/or products brushes, steel wool, glass paper, blades or similar, to remove stains or residues;
- do not machine wash;
- do not soak the product.

The footwear MUST NOT be machine washed or washed by being immersed in or put under water. To clean the lower lateral surface of the sole use soft sponges dampened in warm water not exceeding 30° C, and neutral soap. Do not dry the wet or soaked product in front of heat sources. If the product gets accidentally wet, let it dry away from heat sources and in a dry environment. Optima Molliter Srl shall NOT be held liable for damages caused by abrasive products, by products which attack the materials, by direct exposure to heat sources and by improper use/maintenance.

Class I medical device according to Regulation (EU) 2017/745

SIZES

	40	41	42	43	44	45	46	47
A	26,8	27,5	28,2	28,9	29,6	30,3	31	31,7
B	9,5	9,7	9,9	10,1	10,3	10,5	10,7	10,9
C	26,7	27,2	27,7	28,2	28,7	29,2	29,7	30,2

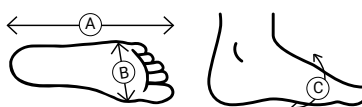

Optima Molliter Srl

Company Certified under: ISO 9001:2015 IT319043 - ISO 13485:2016 IT319042
Via Breda 19/21 62012 Civitanova Marche (MC) Italy T +39 0733 801060
info@optimamolliter.com / www.optimamolliter.com

