YOUR DAILY ACTIVITY

# MANUALE D'USO

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## INTRODUCTION

Optima Molliter Srl declares that YDA footwear is a medical device if used with custom-made plantar insoles made and controlled applying standard tests by licensed healthcare professionals. Conceived and designed in accordance with the state of the art, YDA footwear has been tested and made compliant to the EU Regulation 2017/745. Nevertheless, it should be considered that inappropriate use of the product or underestimation of resulting hazards can cause injuries or harm.

It is important to read this User Manual carefully before using the device and to report any serious incident occurring in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is located.

## MEDICAL USE

**YDA THERAPEUTIC INDICATIONS** *With prescription or medical indication, YDA can be used for the following therapeutic purposes:*

- Treatment of the most common forefoot pains.
- Biomechanical movement problems.
- Rehabilitation after minor forefoot surgery such as bunion, hammer toes after using post surgery orthosis until stabilization.
- Diabetic foot at no risk; at risk 0; at risk 1 only if used together with a custom-made plantar insole, made and controlled by applying standard test by a licensed healthcare professional.
- Metatarsalgia.
- Polyarthritis.
- Morton's Neuroma.
- Arthrosis, Osteoarthritis, only if used with a custom-made plantar insole, made and controlled applying standard tests by a licensed healthcare professional.

**YDA PRO THERAPEUTIC INDICATIONS** *With prescription or medical indication, YDA PRO can be used for the following therapeutic purposes:*

- Treatment of the most common forefoot pains.
- Biomechanical movement problems and rehabilitation management after minor forefoot surgery (such as bunion, hammer toes, after using post-operative orthosis and until stabilization.
- Diabetic foot at no risk; at risk 0; at risk 1, 2, 3 with a custom-made plantar insole, made and controlled by applying standard test by a licensed healthcare professional.
- Metatarsalgia, Polyarthritis, Morton's Neuroma, Arthrosis, Osteoarthritis, with a custom-made plantar insole made and controlled applying standard tests by a licensed healthcare professional.

## CONTRAINDICATIONS

The use of all devices without socks is not recommended. Never use devices without custom-made plantar insoles made and controlled by licensed healthcare professionals, where prescribed and for therapeutic purposes as specified in the MEDICAL USE.

## INSTRUCTIONS FOR USE

**YDA** products can be used directly, or they can accommodate a suitable orthopedic insole made by a qualified health worker, replacing the one supplied as standard in the shoe. Pay particular attention to wearing the **YDA**, taking care to open it, gently moving the slider of the elastic laces outwards, and to insert the foot without applying pressure to the buttress (rear part) of the heel. It is advisable to always use a shoe horn. It is important not to modify the structure and the foreseen support points in order not to alter the functionality of the product. Note that **YDA** footwear uppers and linings materials are dermo-compatible and not chemically treated. Loss of color and/or spots can occur as normal foot perspiration and are to be considered as totally natural. Storage in a cool and dry place and without compressions that could damage its physical features.

## IMPROPER USE

Improper use means using the product in the following conditions:

- Non-compliance with the manufacturer's instructions.
- Non-compliance with current safety regulations.
- For healing purposes without medical prescription/indication.
- For healing purposes without a personalized or custom-made plantar insole made and controlled by applying standard test from a licensed healthcare professionals.
- Sizes alterations.
- Structure changes with elements modification or addition.
- Shoe fitting adaptation with shape alteration.
- Cleaning with wrong products and/or methods.
- Wrong storage.
- People with plantar lesions.
- People allergic to components/treatments/materials used to make the footwear.
- Wearing the product with foot osteo-structural alterations and/or concomitants metabolic/neurological/vascular pathologies without medical prescription/ indication.
- Wearing the product on slippery/oily floors or substances.

## PRODUCT CLEANING

The cleaning of the product must be carried out using:

- For the upper in fabric, use a cloth slightly dampened with water and neutral soap, rub gently and dry.
- For leathers: special creams (moisturizing-polishing) specific for the type of leather to be treated and compliant and approved by the CEE, coated and subsequently polished with a clean, soft and non-abrasive dry cloth.
- For suede or similar: use only brushes or soft cloths and only with dry leather to remove any stains (mud or similar).
- Make sure to perform a regular hydration of the leather by applying special creams, especially in case of particularly soft leathers such as nappa, deer etc.
- Do not use alcoholic and corrosive liquids, detergents, alcohol, laundry detergents, bleach, ammonia and solvents.

- Do not use abrasive materials and/or products brushes, steel wool, glass paper, blades or similar, to remove stains or residues.

The footwear **MUST NOT** be machine washed or washed by being immersed in or put under water. To clean the lower lateral surface of the sole use soft sponges dampened in warm water not exceeding 30°C, and neutral soap. Do not dry wet or soaked products in front of heat sources.

This may irreparably damage the footwear and the warranty will be void. If the product gets accidentally wet, let it dry away from heat sources and in a dry environment. Optima Molliter Srl shall **NOT** be held liable for damages caused by abrasive products, by products which attack the materials, by direct exposure to heat sources and by improper use/maintenance.

## **DISPOSAL OF THE PRODUCT**

The materials used in **YDA** and **YDA PRO** footwear are partially environmentally friendly, and, therefore, must be disposed of in accordance with specific recommendations. Dispose of product responsibly. Refer to the local regulations on the disposal of inert products.

## **WARRANTY**

The warranty refers only to materials used and to manufacturing defects and protects the shoe in accordance with the law only if recognized as defective by the Company.

For the article, which is assumed to be defective, a complaint can only be requested by fulfill the "Complaints Form" which can be requested to the Company and sent to [resi@optimamolliter.com](mailto:resi@optimamolliter.com) only in the presence of:  
- intact, unused article kept in its original packaging and joined by proof of purchase which must not be more than 10 days prior to the claim.

With the prior authorization of Optima Molliter S.r.l., the item must be sent in the original packaging together with the proof of purchase, exclusively carriage paid and indicating the return authorization number communicated by Optima Molliter on the package. Returns that should arrive without affixing the authorization number clearly visible on the package will be rejected. Items used or shipped without compliance with the above specifications, or valued not covered by warranty, may be returned to the sender only at his charge. In the event of any recognized liability of the Company, the product will be returned by the Company to the reseller or customer.

## **DURATION OF USE**

The product maintains its characteristics when it is used and stored correctly throughout its service life according to the environmental characteristics. How you walk and the prolonged use of the shoes may affect their characteristics over time, so it is not possible to predefine how long they will last. In any case, it is advisable to alternate shoes during the week and replace them immediately if signs of wear or damage are found. Using shoes that are not intact may damage tissues over time and affect your posture. Before each use, carefully inspect the shoes and do not use them if there are obvious signs of wear or if they are not perfectly intact.

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