MIAC
MY ACTIVE CARE

MANUALE D'USO

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MIAC
MY ACTIVE CARE

INSTRUCTIONS FOR USE

Class I medical device according to Regulation (EU) 2017/745

INTRODUCTION

Optima Molliter Srl declares that **MAC123** footwear, conceived and designed in accordance with the state of the art, is a medical device class I in compliance to the Regulation EU 2017/745, if used with personalized or customized insoles made and controlled applying standard tests by licensed healthcare professionals for the management of the alterations of the morphology of the foot. **MAC123** footwear is in compliance with the International Guidelines of the IWGDF for the management of the diabetic foot. <https://iwgdfguidelines.org/guidelines/guidelines>. It is important to read this User Manual carefully before using the device and to report any serious incident occurring in relation to the device to the manufacturer and to the competent authority of the Member State in which the user and/or patient is located.

INTENDED USE

The **MAC 1**, **MAC 2** and **MAC 3** lines, if used for containing of customized plantar insoles released under medical indication, made and controlled applying standard tests by a licensed healthcare professionals, are medical devices for the treatment of the foot morphology alterations as well as like following detailed:

MAC1

Protective footwear with semi-rigid midsole for movement control.

It is indicated in case of:

- metatarsal pain;
- polyarthritis;
- Morton's neuroma, with a custom-made plantar insole made by a licensed healthcare professional;
- arthrosis;
- diabetic foot at risk 0 (no loss of protective sensation, no peripheral arterial disease, no deformity);
- diabetic foot at risk 1 (with loss of protective sensation or peripheral arterial disease or deformity).

MAC2

Therapeutic shoe with rigid midsole that avoids forefoot hyper-loading.

Not equipped with insole as it is predisposed exclusively to accommodate custom insoles.

It is indicated in case of:

- diabetic foot a risk 3 (with loss of protective sensation and/or peripheral arterial disease and/or foot deformity and one or more of the following: history of a foot ulcer, a lower-extremity amputation - minor or major, end-stage renal disease);
- arthrodesis of the toes;
- arthrodesis of the tarsus;
- Charcot foot, or biomechanical problems of the movement (high footwear MAC 2).

All **MAC2** footwear must not be used without a custom-made plantar insole made by a licensed healthcare professional.

Any other use is expressly improper and forbidden.

MAC3

Protective footwear with propulsive semi-rigid midsole that redistributes plantar pressure loads.

It is indicated in case of:

- diabetic foot at risk 1 (foot with loss of protective sensitivity or peripheral arterial disease);
- biomechanical problems of the movement;
- metatarsal pain, with a customized plantar insole made by a licensed healthcare professional; If associated with a customized plantar insole MAC3 is indicated for the management of diabetic foot;
- diabetic foot at risk 2 (foot with loss of protective sensitivity associated with deformity or foot with peripheral arterial disease associated with deformity);
- diabetic foot at risk 3 (foot with previous ulcerations) at risk of remission.

CONTRAINDICATIONS

The use of all devices without socks is not recommended.

Never use devices without custom-made insoles made and controlled by licensed healthcare professionals, where prescribed and for therapeutic purposes.

DURATION OF USE

The product maintains its characteristics when correctly used and stored throughout its life cycle according to environmental characteristics. The type of walking and prolonged use of the shoe can change its characteristics over time, so it is not possible to predefine its duration. In any case, it is advisable to alternate shoes during the week and replace them immediately in case of obvious wear or damage. The use of non-integrated footwear can damage tissue and change posture over time. Before each use, carefully inspect the footwear and do not use if it shows obvious signs of wear or imperfect integrity.

INSTRUCTIONS FOR USE

MAC 1 and **MAC 3** products can be used directly or can accommodate a suitable orthopedic plantar insole made by licensed healthcare professionals, replacing the one supplied as standard in the shoe.

The **MAC 2** product can only be used on medical prescription (for footwear with a rigid rocker sole) and only with a custom-made orthopedic plantar insole made by licensed healthcare professionals.

It is important not to modify the structure and the foreseen support points in order to not alter the functionality of the product.

Note that footwear uppers and linings materials are dermo-compatible and not chemically treated.

Loss of color and / or spots can occur as normal foot perspiration and are to be considered as totally natural.

The use of socks is strongly recommended for both winter and summer articles. Storage in a cool and dry place and without compressions that could damage its physical features.

IMPROPER USE

Improper use means using the product in the following conditions:

- Non-compliance with the manufacturer's instructions.
- Non-compliance with current safety regulations.
- For healing purposes without a custom-made insole made and controlled by applying standard tests from a licensed healthcare professional.
- For healing purposes without medical prescription/indication.
- For MAC2, without a personalized plantar insole made and controlled by applying standard tests from a licensed healthcare professional.
- Sizes alterations.
- Structure changes with elements modification or addition.
- Shoe fitting adaptation with shape alteration.
- Cleaning with wrong products and / or methods.
- Wrong storage.
- People with plantar lesions.
- People allergic to components/ treatments/materials used of footwear.
- Wearing the product with foot osteo-structural alterations and/or concomitants metabolic/neurological/vascular pathologies without medical prescription / indication.
- Wearing the product on slippery/oily floors or substances.

CLEANING OF THE PRODUCTS

The cleaning of the product must be carried out using:

- for the upper in fabric, use a cloth slightly dampened with water and neutral soap, rub gently and dry;
- for leathers: special creams (moisturizing-polishing) specific for the type of leather to be treated and compliant and approved by the EEC, coated and subsequently polished with a clean, soft and non-abrasive damp cloth;
- for suede or similar: use only brushes or soft cloths and only with dry skin to remove any stains (mud or similar);
- make sure to perform a regular hydration of the leather by applying special creams, especially in the case of particularly soft leathers such as nappa, deer etc.
- do not use alcoholic and corrosive liquids, detergents, alcohol, laundry detergents, bleach, ammonia and solvents;
- do not use abrasive materials and/or products brushes, steel wool, glass paper, blades or similar, to remove stains or residues;
- do not machine wash;
- do not soak the product.

The footwear **MUST NOT** be machine washed or washed by being immersed in or put under water. To clean the lower lateral surface of the sole use soft sponges dampened in warm water not exceeding 30° C, and neutral soap.

Do not dry the wet or soaked product in front of heat sources. If the product gets accidentally wet, let it dry away from heat sources and in a dry environment. Optima Molliter Srl shall NOT be held liable for damages caused by abrasive products, by products which attack the materials, by direct exposure to heat sources and by improper use/maintenance.

DISPOSAL OF THE PRODUCT

The materials used in MAC 123 footwear are partially environmentally friendly and, therefore, must be disposed of in accordance with specific recommendations. Dispose of product responsibly.

Refer to the local regulations on the disposal of inert products.

WARRANTY

The warranty refers only to materials used and to manufacturing defects and protects the shoe in accordance with the law only if recognized as defective by the Company. For the article, which is assumed to be defective, a complaint can only be requested by fulfilling the "Complaints Form" which can be requested to the Company and sent to resi@optimamolliter.com only in the presence of:

- intact, unused article kept in its original packaging and joined by proof of purchase which must not be more than 10 days prior to the claim.

With the prior authorization of Optima Molliter S.r.l., the item must be sent in the original packaging together with the proof of purchase, exclusively carriage paid and indicating the return authorization number communicated by Optima Molliter on the package. Returns that should arrive without affixing the authorization number clearly visible on the package will be rejected. Items used or shipped without compliance with the above specifications, or valued not covered by warranty, may be returned to the sender only at his charge. In the event of any recognized liability of the Company, the product will be returned by the Company to the reseller or customer.