

TECHNICAL INFORMATION

YDA / VAULT YDAW15 CB



TECHNOLOGY

1. PADDED STABILIZING HEEL COUNTER
 2. SOFT UPPER WITH PROTECTIVE LINING
 3. EASY-LACE: SEMI-ELASTIC CLOSURE WITH CURSOR
 4. ULTRALIGHT ROCKER SOLE WITH ANTI-SLIP INSERT
 5. FOOT ROLLING POINT BEFORE METATARSAL AREA
 6. EXTRA GRIP
- REMOVABLE ANTI-SHOCK FOOTBED
BIOGELLY SYSTEM SHOCK ABSORBING
SPRING SYSTEM® FOR PROPULSIVE PERFORMANCES
OPTIMIZED FIT



YDA footwear stimulates muscles and microcirculation, improves posture, protects the foot from the most common injuries and pains. The technology and construction that characterize the line make it a high-performance product in biomechanical terms, thanks to the propulsion and stability of the movement.

YDA is part of the Optima Molliter Health Program and is in compliance with the IWGDF International Guidelines on the management of diabetic foot. All footwear and devices are tested in the best institutes and certified. By choosing YDA, embrace a conscious and well-being-oriented lifestyle. Our vision is to make people free to move in their world.

THERAPEUTIC INDICATIONS

- Plantar pain (metatarsalgia, heel pain, fasciitis, heel spurs, etc.).
- Polyarthrititis, Morton's neuroma, arthrosis, osteoarthritis when used with custom insoles.
- Fatigue relief and well-being.
- Foot rehabilitation for biomechanics and posture.
- Moderate instability.
- Mild deformities (hallux valgus or varus/hammer toes, etc.).
- Post-trauma rehabilitation, post-minor forefoot surgery (hallux valgus, hammer toes) until stabilization, after use of a brace.
- Contralateral to devices.
- Edematous foot.
- Sensitive and/or rheumatic foot.
- Adjuvant to venous return also indicated in combination with graduated compression stockings.
- Indicated for insertion of custom-made insoles, soft insoles, or devices for walking.
- Diabetic foot, not at risk, risk 0; risk 1, 2*, 3*; *in combination with insoles made by a qualified orthopedist based on an impression. Risk 0 (no obvious foot problems, low risk). Risk 1 (neuropathic foot, low risk). Risk 2 (risky interaction, low risk). Risk 3 (high risk).
- Obesity.

TECHNICAL CHARACTERISTICS

OUTSOLE: EVA WITH RUBBER INSERTS AND INSIDE POLYPROPYLENE INSERT
MIDSOLE: BIOGEL SYSTEM HEEL SHOCK ABSORBER
INSOLE: MEMORY FOAM
HIGH HEEL LAST MEASURE: 15 MM
OUTSOLE DROP MEASURE: 8 MM
UPPER: MESH TEXTILE WITH TPU INSERTS
BACK HEEL: SUEDE LEATHER
LINING: MESH TRANSPIRANT TEXTILE
TONGUE: MESH TEXTILE WITH TPU INSERTS
TONGUE FOAM: FOAM RUBBER
COLLAR FOAM: FOAM RUBBER
BACK COUNTER: MOLDED NYLON
BACK COUNTER FOAM: FOAM RUBBER
REINFORCEMENTS: GLASS WOOL
HARD-TOE END: REBOUND EFFECT NYLON
LACING SYSTEM: EASYLACE
AVERAGE SHOE WEIGHT SIZE 37: Gr. 300
AVERAGE SHOE WEIGHT SIZE 41: Gr. 400
SIZE - FITTING: 41 FIT 11

PRODUCT CLEANING

The cleaning of the product must be carried out using:


- For the upper in fabric, use a cloth slightly dampened with water and neutral soap, rub gently and dry.
- For leathers: special creams (moisturizing-polishing) specific for the type of leather to be treated and compliant and approved by the CEE, coated and subsequently polished with a clean, soft and non-abrasive dry cloth.
- For suede or similar: use only brushes or soft cloths and only with dry leather to remove any stains (mud or similar).
- Make sure to perform a regular hydration of the leather by applying special creams, especially in case of particularly soft leathers such as nappa, deer etc.
- Do not use alcoholic and corrosive liquids, detergents, alcohol, laundry detergents, bleach, ammonia and solvents.
- Do not use abrasive materials and/or products brushes, steel wool, glass paper, blades or similar, to remove stains or residues.
- The footwear MUST NOT be machine washed or washed by being immersed in or put under water.
- To clean the lower lateral surface of the sole use soft sponges dampened in warm water not exceeding 30°C, and neutral soap.
- Do not dry wet or soaked products in front of heat sources. This may irreparably damage the footwear and the warranty will be void. If the product gets accidentally wet, let it dry away from heat sources and in a dry environment.

Optima Molliter Srl shall NOT be held liable for damages caused by abrasive products, by products which attack the materials, by direct exposure to heat sources and by improper use/maintenance.

Class I medical device according to Regulation (EU) 2017/745

SIZES

	35	36	37	38	39	40	41	42	43	44	45	46	47
A	23,36	24,03	24,70	25,37	26,04	26,63	27,30	27,97	28,64	29,31	29,98	30,65	31,32
B	8,20	8,35	8,50	8,65	8,80	8,95	9,10	9,25	9,40	9,55	9,70	9,85	10,00
C	23,10	23,55	24,00	24,45	24,90	25,35	25,80	26,25	26,70	27,15	27,60	28,05	28,50

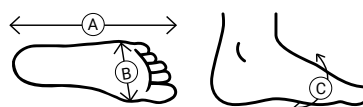

11 E	SEMI-RIGID SOLE
----------------	---------------------------

Optima Molliter Srl

Company Certified under: ISO 9001:2015 IT346378 - ISO 13485:2016 IT346377
 Via Breda 19/21 62012 Civitanova Marche (MC) Italy T +39 0733 801060
 info@optimamolliter.com / www.yda.it / www.optimamolliter.com

