

TECHNICAL INFORMATION

YDA DEDALO / LINK MESH



TECHNOLOGY

1. POWDERED SOFT FOOTBED REMOVABLE AND WASHABLE.
POLYETHER / EXTRA LIGHT / MEMORY REBOUND / ANTIBACTERIAL
2. EASY TO WEAR THANKS TO THE ADJUSTABLE VELCRO CLOSURE
3. BIOMECHANICAL ROCKER BOTTOM IN ORTHOABSORBENT ECO-PU
4. FOOT ROLLING POINT BEFORE METATARSAL AREA
STROBEL COSTRUCTION



YDA DEDALO is a collection of extremely multipurpose and comfortable sandals and slippers. DEDALO has an innovative biomechanical concept, designed in every detail by YDA LAB Research: the removable insole, the scented ortho-absorbent insole plantar and every protective finish. The ergonomic bottom is designed to absorb shock pressure, stabilize posture and make every movement comfortable without sacrificing a contemporary design look.

Optima Molliter Srl declares that YDA DEDALO items are medical devices and can be used for the containment of personalized or custom-made insoles. YDA DEDALO is part of the Optima Molliter Health Program. All footwear and devices are tested in the best Institutes and certified. By choosing YDA DEDALO you embrace a conscious and wellness-oriented lifestyle. Our vision is to make people free to move in their own world.

THERAPEUTIC INDICATIONS

- Plantar pain (metatarsalgia, heel pain, fasciitis, heel spurs, etc.).
- Polyarthritis, Morton's neuroma, arthrosis, osteoarthritis when used with custom insoles.
- Fatigue relief and well-being.
- Foot rehabilitation for biomechanics and posture.
- Moderate instability.
- Mild deformities (hallux valgus or varus/hammer toes, etc.).
- Post-trauma rehabilitation, post-minor forefoot surgery (hallux valgus, hammer toes) until stabilization, after use of a brace.
- Contralateral to devices.
- Edematous foot.
- Sensitive and/or rheumatic foot.
- Adjuvant to venous return also indicated in combination with graduated compression stockings.
- Indicated for insertion of custom-made insoles, soft insoles, or devices for walking.
- Diabetic foot, not at risk, risk 0; risk 1, 2*, 3*; *in combination with insoles made by a qualified orthopedist based on an impression.
Risk 0 (no obvious foot problems, low risk).
Risk 1 (neuropathic foot, low risk).
Risk 2 (risky interaction, low risk).
Risk 3 (high risk).
- Obesity.

TECHNICAL CHARACTERISTICS

OUTSOLE: ECO PU ULTRALIGHT ANTISLIP
MIDSOLE: REMOVABLE SHAPED EXTRALIGHT INSOLE
INSOLE: FELT BIOMECHANICAL STROBEL CONSTRUCTION
HIGH HEEL LAST MEASURE: 5 MM
OUTSOLE DROP MEASURE: 12 MM
UPPER: SOFT ASSEMBLED MESH
LACING SYSTEM: DOUBLE ADJUSTABLE VELCRO
AVERAGE SHOE WEIGHT SIZE 37: Gr. 300
AVERAGE SHOE WEIGHT SIZE 41: Gr. 400
SIZE - FITTING: 41 FIT 11

PRODUCT CLEANING

The cleaning of the product must be carried out using:


- For the upper in fabric, use a cloth slightly dampened with water and neutral soap, rub gently and dry.
- For leathers: special creams (moisturizing-polishing) specific for the type of leather to be treated and compliant and approved by the CEE, coated and subsequently polished with a clean, soft and non-abrasive dry cloth.
- For suede or similar: use only brushes or soft cloths and only with dry leather to remove any stains (mud or similar).
- Make sure to perform a regular hydration of the leather by applying special creams, especially in case of particularly soft leathers such as nappa, deer etc.
- Do not use alcoholic and corrosive liquids, detergents, alcohol, laundry detergents, bleach, ammonia and solvents.
- Do not use abrasive materials and/or products brushes, steel wool, glass paper, blades or similar, to remove stains or residues.
- The footwear MUST NOT be machine washed or washed by being immersed in or put under water.
- To clean the lower lateral surface of the sole use soft sponges dampened in warm water not exceeding 30°C, and neutral soap.
- Do not dry wet or soaked products in front of heat sources. This may irreparably damage the footwear and the warranty will be void. If the product gets accidentally wet, let it dry away from heat sources and in a dry environment.

Optima Molliter Srl shall NOT be held liable for damages caused by abrasive products, by products which attack the materials, by direct exposure to heat sources and by improper use/maintenance.

Class I medical device according to Regulation (EU) 2017/745

SIZES

	35	36	37	38	39	40	41	42	43	44	45	46
A	22,96	23,63	24,30	24,97	25,64	26,31	26,83	27,50	28,17	28,84	29,51	30,18
B	8,80	8,95	9,10	9,25	9,40	9,55	9,75	9,90	10,05	10,20	10,35	10,50
C	24,70	25,15	25,60	26,05	26,50	26,95	27,15	27,60	28,05	28,50	28,95	29,40

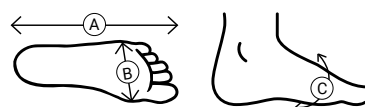

EU 11	SEMIFLEXIBLE SOLE
-------	-------------------

Optima Molliter Srl

Company Certified under: ISO 9001:2015 IT346378 - ISO 13485:2016 IT346377
Via Breda 19/21 62012 Civitanova Marche (MC) Italy T +39 0733 801060
info@optimamolliter.com / www.yda.it / www.optimamolliter.com

