

TECHNICAL INFORMATION

YDA SUPER PRO / YUMA

YDA SUPER PRO

TECHNOLOGY

1. PADDED AND STABILIZING HEEL COUNTER
 2. SHOE UPPER WITH TOTAL COMFORT INNER LINING
 3. BREATHABLE TECH FABRIC
 4. ULTRALIGHT ROCKER SOLE ANTI-SLIP INSERTS
 5. FOOT ROLLING POINT BEFORE METATARSAL AREA
 6. CLOSURE ADJUSTABILITY
- SPRING SYSTEM® PRO BASE FOR PROPULSIVE PERFORMANCE
REMOVABLE ANTISHOCK FOOTBED
RIGID MIDSOLE COVERED WITH D-COVER WHICH REDISTRIBUTES PRESSURE LOADS



YDA SUPER PRO with propulsive **rigid** sole, was born from the enhancement of YDA technologies to maximize stabilization, active control of the movement and to guarantee an extremely comfortable fit. The extra fit (**FIT 14**) makes the product suitable for larger custom-made insoles or orthotics made by specialized healthcare professionals. Designed with a biomechanical rocker sole with an early rolling point, YDA SUPER PRO facilitates step-off and allows for improved gait function. Optima Molliter Srl declares that YDA SUPER PRO footwear is a medical device and can be used to support customized or custom-made insoles. YDA SUPER PRO is part of the Optima Molliter Health Program and is in compliance with the IWGDF International Guidelines on the management of diabetic foot. All footwear and devices are tested in the best institutes and certified. By choosing YDA SUPER PRO, embrace a conscious and well-being-oriented lifestyle. Our vision is to make people free to move in their world.

THERAPEUTIC INDICATIONS

- Plantar pain (metatarsalgia, heel pain, fasciitis, heel spurs, etc.).
- Polyarthritis, Morton's neuroma, arthrosis, osteoarthritis when used with custom insoles.
- Fatigue relief and well-being.
- Foot rehabilitation for biomechanics and posture.
- Moderate instability.
- Mild deformities (hallux valgus or varus/hammer toes, etc.).
- Post-trauma rehabilitation, post-minor forefoot surgery (hallux valgus, hammer toes) until stabilization, after use of a brace.
- Contralateral to devices.
- Edematous foot.
- Sensitive and/or rheumatic foot.
- Adjuvant to venous return also indicated in combination with graduated compression stockings.
- Indicated for insertion of custom-made insoles, soft insoles, or devices for walking.
- Diabetic foot, not at risk, risk 0; risk 1, 2*, 3*; *in combination with insoles made by a qualified orthopedist based on an impression.
Risk 0 (no obvious foot problems, low risk).
Risk 1 (neuropathic foot, low risk).
Risk 2 (risky interaction, low risk).
Risk 3 (high risk).
- Charcot foot.
- Obesity.

TECHNICAL CHARACTERISTICS

OUTSOLE: EVA WITH RUBBER INSERTS AND INSIDE POLYPROPYLENE INSERT
MIDSOLE: SHOCK ABSORBING FOAM INSOLE
INSOLE: MEMORY FOAM
HIGH HEEL LAST MEASURE: 15 MM
OUTSOLE DROP MEASURE: 8 MM
UPPER: ASSEMBLED NYLON AND LEATHER
BACK HEEL: TECHNO MATERIAL
LINING: MESH TRANSPIRANT TEXTILE
TONGUE: ASSEMBLED NYLON
TONGUE FOAM: FOAM RUBBER
COLLAR FOAM: FOAM RUBBER
BACK COUNTER: MOLDED NYLON
BACK COUNTER FOAM: FOAM RUBBER
REINFORCEMENTS: GLASS WOOL
HARD-TOE END: REBOUND EFFECT NYLON
LACING SYSTEM: ADJUSTABLE DOUBLE VELCRO
AVERAGE SHOE WEIGHT SIZE 37: Gr. 400
AVERAGE SHOE WEIGHT SIZE 41: Gr. 400
SIZE - FITTING: 41 FIT 14

PRODUCT CLEANING

The cleaning of the product must be carried out using:


- For the upper in fabric, use a cloth slightly dampened with water and neutral soap, rub gently and dry.
- For leathers: special creams (moisturizing-polishing) specific for the type of leather to be treated and compliant and approved by the CEE, coated and subsequently polished with a clean, soft and non-abrasive dry cloth.
- For suede or similar: use only brushes or soft cloths and only with dry leather to remove any stains (mud or similar).
- Make sure to perform a regular hydration of the leather by applying special creams, especially in case of particularly soft leathers such as nappa, deer etc.
- Do not use alcoholic and corrosive liquids, detergents, alcohol, laundry detergents, bleach, ammonia and solvents.
- Do not use abrasive materials and/or products brushes, steel wool, glass paper, blades or similar, to remove stains or residues.
- The footwear **MUST NOT** be machine washed or washed by being immersed in or put under water.
- To clean the lower lateral surface of the sole use soft sponges dampened in warm water not exceeding 30°C, and neutral soap.
- Do not dry wet or soaked products in front of heat sources. This may irreparably damage the footwear and the warranty will be void. If the product gets accidentally wet, let it dry away from heat sources and in a dry environment.

Optima Molliter Srl shall NOT be held liable for damages caused by abrasive products, by products which attack the materials, by direct exposure to heat sources and by improper use/maintenance.

Class I medical device according to Regulation (EU) 2017/745

SIZES

	35	36	37	38	39	40	41	42	43	44	45	46	47
A	23,26	23,93	24,60	25,27	25,94	26,53	27,20	27,87	28,54	29,21	29,88	30,55	31,22
B	8,20	8,35	8,50	8,65	8,80	9,05	9,20	9,35	9,50	9,65	9,80	9,95	10,10
C	24,90	25,35	25,80	26,25	26,70	27,15	27,60	28,05	28,50	28,95	29,40	29,85	30,30

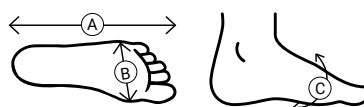

Fit 14	RIGID SOLE
--------	------------

Optima Molliter Srl

Company Certified under: ISO 9001:2015 IT346378 - ISO 13485:2016 IT346377
Via Breda 19/21 62012 Civitanova Marche (MC) Italy T +39 0733 801060
info@optimamolliter.com / www.yda.it / www.optimamolliter.com

